## Manual

PennyTalks (PT) The personal digital voice labeler with speech supported user instructions and real-time backup.

## Quick guide:

3

well.

selected.

Put a self-adhesive label to whatever you would like to memorize.

Hold the sensor to the label and record your own voice memo in PT.

Playback your voice memo by holding the sensor of PT to the pre-recorded label.

Switch on/switch off:

Insert the batteries. Press and hold the green

If PT is left without any activity for more than 120

How to set the language of the

speech supported user instructions:

PennyTalks holds 18 pre-stored language modes for

1. Touch to any of the languages available on the

2. PT reports choosen language and which is now

language setting chart with the sensor.

button on/off button for 3 seconds.

seconds, it will automaticly switch off.

the speech supported user instructions.



safe to wash and to tumble dry

Language setting chart

## **Playback:**

Hold the sensor to the label. PT now reports your recorded memo.